Instruction Manual
Manuale
Manual
Manuel d'utilisation
Bedienungsanleitung
Handleiding

CLARITY™
Amplifying Your Life™

Amplifier

ECE125

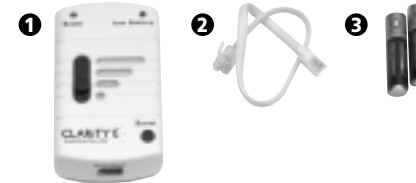
Importer:
Plantronics B.V.
Southpoint Building C
Scorpius 140
2132 LR Hoofddorp
The Netherlands

© 2003 Clarity™, a Division of Plantronics, Inc.
All rights reserved. Clarity and Amplifying Your Life
are trademarks or registered trademarks
of Plantronics, Inc.

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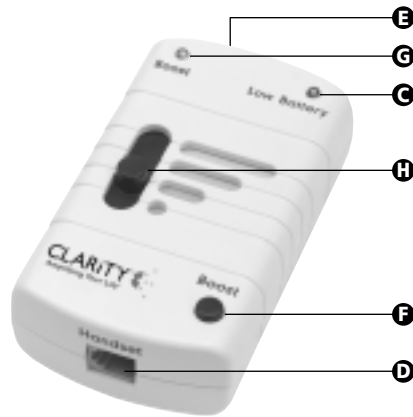
www.clarityproducts.com

Introduction to the ECE125 amplifier

Thank you for choosing the Clarity™ amplifier ECE125. Before you use this product, we advise you to read this instruction manual thoroughly. You will then be familiar with all the functions, capabilities and safety instructions of this amplifier.

A 'quick reference' is incorporated into this manual. This is primarily aimed at the installation and setting of various functions. If you wish to use it, read only the highlighted parts of the text.

Keep this manual near your telephone, so that you can refer to it quickly if necessary.

Service and warranty

Check the contents of the package

Check that the package contains all illustrated products. If this is not the case, contact the retailer where you bought this amplifier.

Warranty

- The following 2 year warranty and service information applies only to product purchased and used in the countries listed on page 49 You may also contact your local retail dealer for warranty information in your country.
- Clarity™ guarantees the satisfactory condition of the equipment with regard to manufacturing and material defects for 2 years from the date of purchase. (Please keep your original receipt safely).
- This product must be installed and used in accordance with the instructions contained in this User Guide, and serviced at an approved Plantronics Service Centre. Failure to comply with these conditions may render the warranty invalid.

2 Year Warranty and Repair Service

- Products under warranty are replaced free of charge.
- All units serviced to full production standards using original factory parts.

Customer Service

If you need support in installing or using your Amplifier ECE125, then please contact the Clarity Customer Service (see page 49).

2

Safety Instructions

Before you use this amplifier, please read the following safety instructions. Please adhere to them strictly to avoid the risk of fire, electric shocks and personal injuries:

- Read all instructions carefully.
- Make note of the warnings and instructions on the product.
- Remove the plug of the telephone from the wall socket before you clean the amplifier.
- Do not use a liquid cleaning detergent or spray can, only a damp cloth.
- Do not use the telephone with the amplifier near water.
- Use the telephone flex and batteries as indicated in this instruction manual. Take empty batteries to a collection point for chemical waste.
- Never throw batteries into a fire; they might explode.

The Clarity™ Amplifier ECE125 complies with the following EC Directives:

- 73/23/EEC Low Voltage Directive.
- 89/336/EEC Electromagnetic Compatibility Directive.
- 93/42/EEC Medical Devices Directive (class 1).

3

Parts of the amplifier

- A** Battery cover
- B** AAA batteries
- C** Battery indicator (red)
- D** Connection to telephone handset
- E** Connection to telephone
- F** Boost button (for additional amplification)
- G** Boost indicator (green)
- H** Tone control

Connecting the amplifier

Installing or replacing batteries (see **A**, **B** and **C**)

1. When you install or replace batteries, first disconnect the amplifier ECE125 from the telephone.
2. Open the battery cover **A** and fit the batteries as indicated **B**.
3. Close the battery cover **A** properly.
4. If the batteries are flat, the red light **C** will not illuminate.
Replace the batteries as soon as possible.
The indicator will only illuminate if the amplifier is in use.

N.B. You need 2 AAA batteries for this amplifier. Battery life ranges from two to six months, depending on the frequency of use of the amplifier and the type of battery. On purchase, we recommend alkaline batteries.

4

Connecting the telephone handset (see **D**)

1. Disconnect the telephone flex with handset from your telephone by removing the modular plug from the telephone.
2. Connect the handset to the amplifier. Connect the telephone flex via of the opening at the bottom of the amplifier **D**.

Connecting the amplifier to the telephone (see **E**)

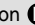
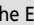
1. Plug in the connection flex **E** with one end in the top of the amplifier **E** and the other end in the telephone (where the handset was previously connected).
2. Pick up the handset and check whether you hear a dial tone.

The Clarity™ Amplifier ECE125 cannot be connected to a telephone with the dialling mode installed in the handset.

5

Using the amplifier

Volume (see and)

The Clarity™ amplifier ECE125 amplifies incoming sound up to 15 decibels as a standard. If you press the Boost button , the ECE125 will provide additional amplification up to 25 decibels. The Boost indicator  then turns green.

The Boost function switches off automatically when you place the handset back on the base. Press the Boost button again for extra loud amplification of your next call.

Tone control (see)

Use the tone button  to control the tone of the incoming voice.

To increase the volume of the high tones, slide the button upwards.

To reduce the volume again, slide the button downwards.

Clarity Power™ technology ensures that only the high tones are amplified. This technology not only makes calls louder, but also easier to understand.

Warning:

If the Boost function  is switched on and/or the tone control  is not at the 'normal' position the volume may be extra loud.

This mode is intended specifically for hearing

6

impaired users and could be harmful to other users. If you want to adjust the tone, slide the control gradually. Avoid prolonged or unnecessary exposure to high volume levels.

Problem solving (solving problems yourself)

If you have problems, always check first whether:

- 1) all flexes are connected correctly;
- 2) the batteries are installed correctly or whether they need to be replaced.

To check whether the problems are caused by the amplifier, plug the telephone handset directly into the telephone again, without the amplifier in between. If this solves the problem, it could be caused by the amplifier. If the problem is not solved, it may be due to the telephone or telephone line. In this case, contact the manufacturer of the telephone or your telephone company.

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